



**3.0** mL  
*Madie*®  
Adjustable multi-dose pen injector

Volume 

Viscosity 

|                         |   |
|-------------------------|---|
| Device Type             | Disposable Pen Injector                         |
| Primary Container       | Up to 3mL Cartridge                             |
| Dosage Type             | Variable   Multi-Dose   10-990µL                |
| Route of Administration | Subcutaneous                                    |
| Needle Attachment       | Manually Attached                               |
| Needle Insertion        | Manual  |
| Activation Method       | Button Activated                                |
| Injection Feedback      | Injection button locks in place after injection |

## Handling Instructions



1 Screw on outer needle cap to attach the needle. Remove the cap



2 Turn dial to the priming dose & remove the inner needle shield



3 Hold device with the needle pointing up. Press injection button to prime



4 Dial to [00] to reset before dialing the required dose



5 Hold against injection site & press button until injection is complete



+ Recap the needle & unscrew. Discard the needle in a sharps container

## Key Features

- ✓ Advanced reset feature
- ✓ Dial-back dose correction
- ✓ Ergonomic & innovative design
- ✓ Dosing increments from 10-990 $\mu$ L