



SDI MIX[®] + NIT[®]

Twist & Mix

Volume 

Viscosity 

Device Type

Disposable Auto Injector

Primary Container

1mL Dual Chamber Cartridge with Pre-Installed Needle (NIT[®])

Dosage Type

Fixed | Full Contents

Route of Administration

Subcutaneous

Needle Insertion

Manual

Activation Method

Needle Cover

Injection Feedback

Audible click at beginning of injection, viewing window

Handling Instructions



1 Turn rear dial to mix



2 Hold device with the cap upright. Twist & remove cap. Needle cover will extend & device will automatically prime



3 Press & hold against injection site



+ Discard in sharps container

Key Features

- ✓ Simple 3-step operation
- ✓ Needle cover activation
- ✓ Manual needle insertion
- ✓ Audible, visual & tactile feedback
- ✓ Needle Isolation Technology (NIT[®])
- ✓ Automatic reconstitution & priming
- ✓ Permanently hidden needle