



VSDI[®] + NIT[®]

Variable Dose Pre-Attached
Needle Solution

Volume 

Viscosity 

Device Type

Auto Injector

Primary Container

1.5mL to 3mL Cartridge with Pre-Installed Needle (NIT[®])

Dosage Type

Single Fixed or Variable Dose

Route of Administration

Subcutaneous

Needle Insertion

Manual

Activation Method

Needle Shield

Injection Feedback

Audible continuous clicks, visual & tactile feedback

Handling Instructions



1 Hold device with cap facing up. Twist & remove needle cap, causing needle shield to automatically extend



2 Dial desired dose



3 Press & hold device against injection site to activate



+ Discard in sharps container

Key Features

- ✓ Simple 3-step operation
- ✓ Needle shield activation
- ✓ Manual needle insertion
- ✓ Continuous audible clicks & visual feedback
- ✓ Needle Isolation Technology (NIT®)
- ✓ Customizable needle length & gauge
- ✓ Simple preparation
- ✓ Permanently hidden needle